

# How to Install 3D SIZE ME Application: A STEP BY STEP GUIDE



## 1. Download 3d size me application

On an iPhone or iPad, go to the app store and download the app: [3DSizeME](#)

## 2. Set up with BMI

Once downloaded, go to the settings, found in the top left corner by dragging to the right

- USER ID: (Enter your company name)
- Bundle ID: **BMI001** (hit the x at the top right and let the app close)
- Once reopened, it should show BMI on the top right
- Go back to settings and turn on display scan preview
- Change Upload target to **EMAIL**
- Enter the following default email: [production@bmiortho.com](mailto:production@bmiortho.com)
- Turn on EMAIL A COPY OF THE ORDER FORM this will allow you to email a PDF to your admin after the scan is submitted as you cannot get the order form from the encrypted file).

You're all set!